MEDISAFE®

STURZMATTE PREMIUM

PREMIUM FALL MAT
VALMAT PREMIUM

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NOTIZEN

Dear User,

Thank you for choosing this product. This user manual describes how to prepare, operate, use, maintain and care for the product. It is a guideline for users and caregivers to ensure the safe and risk-free use of the product. Please read this user manual carefully and keep it readily at hand at all times.

CONTENTS

The product is supplied in a cardboard box. The cardboard box serves for the future storage of the product and must not be discarded. Please first check whether the contents of the box is complete and undamaged. Notify the supplier immediately upon receipt should you detect any damage. Complaints raised at a later point in time cannot be accepted.

Package content

- Storage box
- Fall Mat Premium
- This manual

TECHNICAL SPECIFICATIONS

Embossed, non-slip surface.

Material: Polyurethane (PU) foam

Weight: 3,5 kg

Dimensions: 187 x 60 x 2 cm

Item no. 51223 - colour: brown

Item no. 51224 - colour: grey

INTENDED PURPOSE

The MEDISAFE® Fall Mat Premium is the ideal choice as part of a comprehensive fall prevention program for providing a safe environment for patients at high risk of falling out of bed. High quality foam inner core construction provides maximum absorption and rebound to reduce impact in the event of a fall. The mat's tapered, beveled edges allow equipment to roll over with ease and its non-slip bottom keeps the mat secure to help prevent accidental tripping. The embossed non-slip surface is not only attractive but extremely durable and puncture-resistant. Non-slip pattern helps minimize slipping. When used as a component of a comprehensive fall prevention program, the MEDISAFE® Fall Mat Premium reduces the impact of a fall from bed to help minimize injury. The Joint Commission's Patient Safety Goals regulations and the Centers for Medicare and Medicaid Services guidelines have created increased urgency for the prevention of resident falls. The patient or resident room is an area where falls frequently occur, particularly when getting into or out of bed. In addition to low height beds that have been found to help reduce the incidence of falls; impact reduction MEDISAFE® Fall Mat Premium placed alongside the bed have become a cost effective means to help

reduce the incidence of patient trauma and severity of injury by providing a cushioned, slide resistant surface.

DESIGN

The low profile with tapered edges allows equipment such as overbed tables or patient lifts to be easily rolled over the mat. The non-slip underside means that the mat cannot slip away and there is no risk of caregivers tripping over the mat.

USE

Place the MEDISAFE® Fall Mat Premium next to the bed on the floor. If the bed is in the middle of the room, a fall mat should be used for each side.

**NOTE**

Make sure that the side with the embossed pattern and logo is facing upwards and the non-slip underside is resting on the floor.

INDICATION

The product is used to mitigate/prevent injuries due to a fall event, e.g. from a care bed. The use of the fall mat is indicated for patients or residents who have a higher risk of falling out of bed.

CONTRAINDICATION

The use of the MEDISAFE® Fall Mat is not indicated for persons whose height exceeds 187 cm.

CLEANING AND DISINFECTION

1. All cleaning solutions must be properly diluted according to the manufacturer's instructions. Fall Mats should be thoroughly cleaned after each patient use.
2. Superficial dirt on the cover may be removed by wiping with soft cloth moistened with mild soap or organic cleanser and water. This will be effective on common stains such as blood, urine and perspiration.
3. Cleaning and infection control may be carried out on the fall mat top cover with a sodium hypochlorite solution (0.1% or 1000 parts per million available chlorine), 8–10% bleach/water and/or a EPA-registered Quaternary or Phenolic wipe down can also be used when properly diluted to manufacturer's instructions. Any cleansing agent containing abrasives should not be used as well as any iodophor type disinfectants (Betadine for example) as this will stain the cover. Please allow top cover to completely air dry prior to use or storage as adequate drying is necessary care and maintenance for the top cover coated materials.
4. Wear appropriate protective clothing and gloves when cleaning Fall Mats.

**NOTE**

Always conduct periodic inspections before and during use. Do not use if the cover has holes or tears. A damaged Fall Mat Premium will allow interior foam core to absorb moisture and harbor bacteria, thereby compromising infection control efforts.

**WARNING**

CARELINE gives no guarantee, expressed or implied, that utilizing the MEDISAFE® Fall Mat Premium will prevent trauma to the resident or patient. To reduce the possibility of family members or healthcare staff from tripping on the mat; inform everyone caring for the patient, especially those working at night that the Fall Mat Premium has been placed on the floor next to the bed.

RE-USE

The MEDISAFE® Fall Mat Premium can be re-used after disinfection.

USEFUL LIFE

The expected useful life of the MEDISAFE® Fall Mat Premium is up to four years if used in accordance with this manual and undamaged. It can be used beyond this period if is in a safe condition and does not show any signs of damage.

MAINTENANCE

The MEDISAFE® Fall Mat Premium is maintenance-free.

WARRANTY

The statutory warranty period of two years from date of purchase applies. The warranty covers any defects of the product which are demonstrably due to defects of material or manufacturing. Any damage incurred by natural wear and tear, intent, negligence, or improper operation or use is excluded from the warranty. The warranty also expires if unsuitable detergents are used. In the event of non-compliance with the instructions contained in this user manual, of improper maintenance or unauthorised modifications to the product without the manufacturer's prior written consent, all liability and warranty claims expire with immediate effect.

DISPOSAL

Please dispose of the MEDISAFE® Fall Mat Premium in an environmentally friendly manner via an approved waste disposal company or your local waste disposal facility. Please make sure to comply with any applicable regulations.

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CARELINE GmbH
CARELINE B.V.
CARELINE SAS